



EdgeEndo® Chlorhexidine

EdgeEndo® Chlorhexidine is a 2% Chlorhexidine Gluconate solution in water with surfactant to lower surface tension. The solution is root canal cleanser for use in endodontic procedures.

Indications for use

EdgeEndo® Chlorhexidine is intended to irrigate and cleanse root canal systems after endodontic instrumentation.

Instructions for use

Important: Set rubber stops on irrigation needle tips a minimum 2mm short of working length. Avoid wedging irrigation needle tip into root canal and excessive pressure during irrigation.

Syringes

1. Remove cap from Blue syringe.
2. Attach irrigation needle tip into syringe.
3. Irrigate canal.
4. Discard irrigation needle tip & syringe after use.


Bottle

1. Select desired size syringe and irrigation tip.
2. Fill syringe with EdgeEndo® Chlorhexidine.
3. Attach irrigation needle tip onto the syringe.
4. Irrigate canal.
5. Discard irrigation needle tip & syringe after use.

Distributed By:
EdgeEndo®
Albuquerque, NM 87109
505.872.1115
www.EdgeEndo.com
Made in the USA



Caution, consult
accompanying
documents

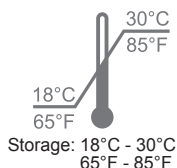

Storage: 18°C - 30°C
65°F - 85°F

Rx only
For Dental Use Only