

**DIRECTIONS FOR USE**

For all EdgeTaper Platinum™ Heat-Treated and EdgeTaper™ (Non-heat Treated) Files use your current or similar speeds, torques and techniques as you are using with ProTaper® and ProTaper Gold®.

**Other Helpful Hints, Tips, and Techniques**

Speed: 300-400 rpm Torque: 4.0-5.2 Ncm  
 EdgeLube™ EDTA: Use 17% EdgeLube™ EDTA Liquid in the Chamber and 17% EdgeGel™ EDTA Gel to coat each file during use.

**Straight/Wider Canals:**

Glide Path

- Fill chamber with EdgeLube™ EDTA Liquid.
- Take #10 hand file to estimated Working Length.
- Establish working length with Apex Locator using handfile or X-ray.

Optional Steps:

- Established canal patency by taking #10 1mm past WL
- Take a #15 hand file to working length.

Shape and Finish Canal

- Fill chamber with EdgeLube™ EDTA liquid
- Take S1 to Working Length
- Rinse with EdgeLube™ EDTA liquid
- Recapitulate #10 hand file to Working Length
- Take S2 to Working Length
- Rinse with EdgeLube™ EDTA liquid
- Recapitulate #10 hand file to Working Length
- Take F1 to Working Length
- Rinse with EdgeLube™ EDTA liquid
- Recapitulate #10 hand file to Working Length
- If a larger file is needed then use F2 & F3
- Disinfect and Obturate Canals with EdgeTaper™ Gutta Percha Points

**Curved/Narrow/Long Canals**

Glide Path in Coronal 2/3

- Fill chamber with EdgeLube™ EDTA liquid
- Take #10 hand file 2/3 down canal

Optional Step:

- Take a #15 hand file 2/3 down canal

Shape Coronal 2/3 of Canal

- Fill chamber with EdgeLube™ EDTA liquid
- Take SX to length of hand files

Form Glide Path in Apical 1/3

- Fill chamber with EdgeLube™ EDTA liquid
- Take #10 hand file to estimated Working Length.
- Establish working length with Apex Locator using handfile or X-ray.


Optional Steps:

- Established canal patency by taking #10 1mm past WL
- Take a #15 hand file to working length

Shape and Finish Apical 1/3

- Fill chamber with EdgeLube™ EDTA liquid
- Take S1 to Working Length
- Rinse with EdgeLube™ EDTA liquid
- Recapitulate #10 hand file to Working Length
- Take S2 to Working Length
- Rinse with EdgeLube™ EDTA liquid
- Recapitulate #10 hand file to Working Length
- Take F1 to Working Length
- Rinse with EdgeLube™ EDTA liquid
- Recapitulate #10 hand file to Working Length
- If a larger file is needed then use F2 & F3
- Disinfect and Obturate Canals with EdgeTaper™ Gutta Percha Points

HEAT-TREATED FIREWIRE™ NiTi

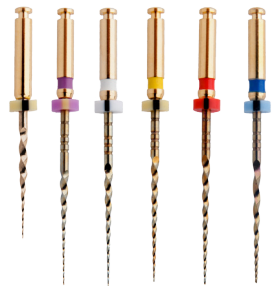

SX S1 S2 F1 F2 F3

**EDGETAPER™**  
PLATINUM

NON HEAT-TREATED NiTi



SX S1 S2 F1 F2 F3

**EDGETAPER™**