

DIRECTIONS FOR USE

For users of Brasseler Sequence® & ESX®, Tulsa Dental Vortex® & Vortex Blue®, Kerr TF Adaptive or K3: Use your current file technique along with the same speeds and torques. Or use the below technique.

1. Take #10 hand file to length.
2. Take EdgeGlidePath™ to length.

If EdgeGlidePath™ feels:

- Tight resistance - Small Canal
- Medium resistance - Medium Canal
- Light resistance - Large Canal

3. Small Canal: 25/04->20/04 repeat until desired file is to length.

Medium Canal: 35/04->30/04 repeat until desired file is to length.

Large Canal: 45/04->40/04 repeat until desired file is to length.

Motor Settings:

EdgeGlidePath™
RPM 300-500 Torque 320g-cm.

EdgeSequel™
RPM 500 Torque 410g-cm All Files

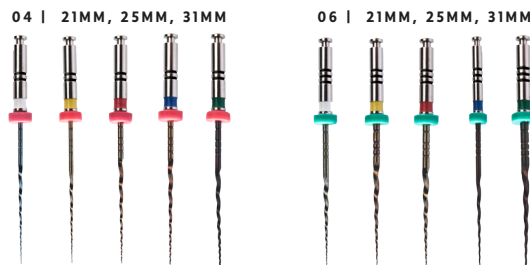
Use EdgeLube™ 17% Liquid EDTA during all hand & rotary file use.

Canal Disinfecting:

- Rinse one minute with EdgeLube™ 17% Liquid EDTA to remove smear layer.
- Rinse 5-10 minutes with full strength NaOCl.
- Rinse one minute with EdgeLube™ 17% Liquid EDTA to rinse out NaOCl.
- Rinse 5-10 minutes with Chlorohexadine.

Canal Obturation:

- Use your current Sealer and 04 or 06 Gutta Percha points or EdgeSeal™ and Edge Gutta Percha™ points.

**EDGESEQUEL™**
SAPPHIRE