

EDGEENDO<sup>TM</sup> Calcium Hydroxide with Barium Sulfate

## INSTRUCTIONS FOR USE

### Water Soluble Temporary Root Canal Filling Material

#### Applications

- Exposed pulp in pulp capping and pulpotomy
- Leakage canal
- Apexification
- Formation of hard tissue barrier
- Root canal filling material

#### Direction for Use

- Clean and dry it after preparing the root canal.
- Set up a disposable tip in syringe and insert the ring provided for an easy re-direction of the tip.
- Disinfect the disposable tip with ethanol.
- Fill the canal completely by pushing the syringe plunger while withdrawing the tip slowly.
- Clean any excess paste with a sterilized cotton pellet.
- Remove the used tip, fix a new one and cover up.

#### How to Use Rotary Ring


- If in contact with air, content could be hardened, so it is necessary to cover a cap on the needle after use.
- Putting a syringe into a rotary ring is more convenient to handle & use the rotary ring as a wrench to tighten or loosen a tip.

#### Advantages

- Easy cleaning and removal.
- Excellent antibacterial effect and radiopacity.
- Premixed paste in a convenient syringe.
- Excellent accessibility to the root canal and prevention of cross-contamination.

#### Contents

- 2.2g paste in a syringe
- Disposable tips
- One ring rotator for the direction control of the tip

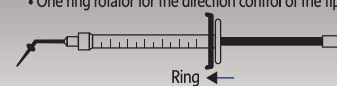

#### Storage

- Do not fill beyond the apex.
- For dental use only.
- Do not use in allergic person.
- Keep syringe capped at all times, when not in use.
- Store in a dry and cool area to avoid drying or separation of the paste.

#### Shelf life

2 years from the manufactured date

#### Symbols



Use By



Batch Code



0120

CE-Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC



Authorized representative in the EC



Consult operating instructions



Manufacturer

Manufactured for:

**EDGEENDO**<sup>®</sup>

7510 Montgomery Blvd NE, Suite 106  
Albuquerque, NM 87109  
phone: 855.985.3636  
www.edgeendo.com

EC REP

Technical & General  
2 Albion Place, London W6 0QT, United Kingdom  
Tel: +44-20-8834-7575  
Fax: +44-20-8834-7570

Made in Korea