

Hydraulic Temporary Restorative



## INSTRUCTIONS FOR USE

### Hydraulic Temporary Restorative

#### Recommended Indication

To be used by a dental professional as a temporary filling material.

#### Contraindication

Contraindicated for patients with known hypersensitivity to hydraulic temporary restoratives. In case of allergy, refer to physician.

#### Components

40g jar individual box  
Box size: 35×35×36mm

#### Directions for use

1. Dry the prepared cavity with a cotton pellet (do not desiccate).
2. Apply the necessary amount of MD- Temp into the cavity with the use of a suitable instrument.
3. Let the patient bite down on the filling to check for interferences in case of Class I cavities. Otherwise, contour filling by adapting it to the cavity walls.
4. Remove excess by wiping it off with a moist cotton pellet towards the margins of the cavity.

#### Notes

Close the container immediately after removal of the material required. Do not mix or contaminate with any other material.


#### Storage

Store at temperatures 10 °C -25 °C (50 °F - 77 °F)

#### Shelf life

2 years from the manufactured date.

#### Symbols

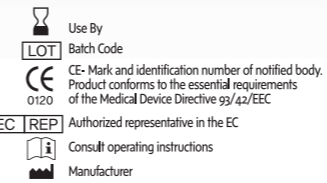

#### Manufactured for:

### EDGEENDO®

7510 Montgomery Blvd NE Suite 106  
Albuquerque, NM 87109  
phone: 855.985.3636  
www.edgeendo.com

#### EC REP

Technical & General  
2 Albion Place, London W6 0QT, United Kingdom  
Tel: + 44- 20- 8834- 7575  
Fax: + 44- 20- 8834- 7570

Made in Korea