





## Technical specifications

Intended use	Endodontics				
Indications for use	Non-surgical endodontic retreatment.				
Description	Titanium-Niobium tip for retreatment in the middle and apical thirds and the extraction of broken instruments				
Tip type	Smooth				
Chemical composition	The alloy of the medical device contains: <ul style="list-style-type: none"><li>• Titanium</li><li>• Aluminium</li><li>• Niobium</li></ul>				
Commercial reference	ET25 : F88018				
Medical class	Class of medical device: IIa according to 93/42/EEC directive				
Adjustments	Newtron range	P5 Range	Suprasson P5 Booster	Prophy Max - P-Max - Implant Center 2 - Piezotome 2	Irrigation
	7	7	5	3	


## Service life

As it is not possible to establish a maximum number of uses (that may be determined by many parameters such as duration of use, hardness of enamel, the force applied, wear), we recommend that medical devices being routinely used are replaced at least once a year. Replace the medical device if the tip does not vibrate at the expected frequency, the treatment is not progressing as normal and is taking longer or is at a standstill.


## Associated documentation

Document title	References
Consulting electronic user instructions	J00007
Cleaning, disinfection and sterilization instructions for tips	J02001
Endo One Kit clinical tips	J08001
Endosuccess Retreatment Kit clinical tips	J08121
Ultrasonic generator power settings table	J58000
General instructions relating to the complete range of standard ultrasonic dental tips	J02101


Sterilization

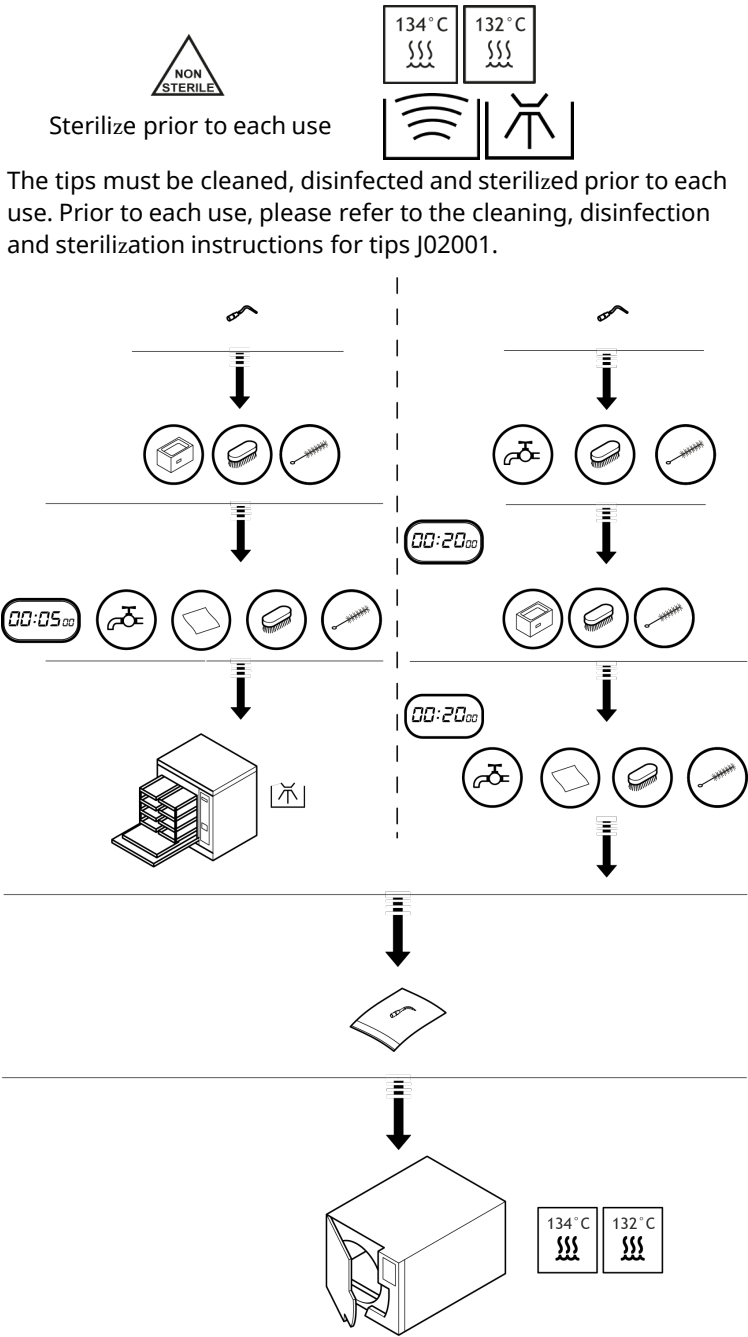


Installation

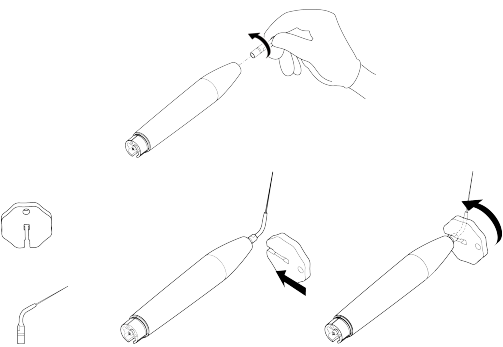

Disassembly

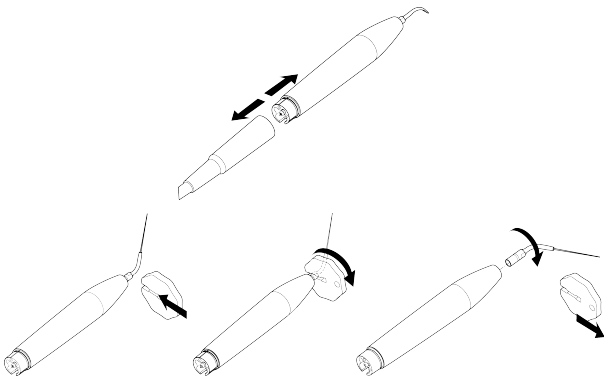

Manufactured For:

 EdgeEndo  
5600 Wyoming Blvd NE  
Suite 100  
Albuquerque, NM 87109

[www.edgeendo.com](http://www.edgeendo.com)



Date of first CE marking

1998

Tip | ET25 | J00721 | (98) | V1 | 03/2017 | NB26EN090A