



EDGEENDO®

EDGEMIX™

EdgeMix™ Indications

EdgeMix™ 2-in-1 solution is used after endodontic instrumentation for the removal of the smear layer and dentin mud, while killing root canal bacteria. EdgeMix™ is an EDTA based formula enhanced with chlorhexidine for added bactericidal properties.

Following the use of EdgeEndo 5% Sodium Hypochlorite, EdgeMix™ removes smear layer and kills bacteria in one easy step.

EdgeMix™ is compatible with sodium hypochlorite (NaOCl) and will not form precipitate.

Instructions for Use:

1. Irrigate with EdgeEndo 5% Sodium Hypochlorite as needed debridement during instrumentation, prior to use of EdgeMix™
2. Fill a clean syringe with EdgeMix™ solution³. Attach an irrigation needle tip securely to syringe (e.g. irrigating tip)
4. Irrigate for 60-90 seconds
5. Irrigate again with EdgeEndo 5% Sodium Hypochlorite for further debridement and disinfection.
6. Finish with a final rinse of EdgeMix™ for optimal canal cleanliness
7. Evacuate the canal with a micro suction tip
8. Blot out the canal using 1-2 paper points
9. Seal and obturate as normal

WARNING

May be harmful if swallowed. Causes mild skin irritation. Causes eye irritation. Wash skin thoroughly after handling. Avoid release into environment. Wear protective gloves/eye wear/face wear. Dispose of contents in accordance with local, state, and federal regulations.

FIRST AID: IF ON SKIN: Wash with plenty of soap and water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. Call a poison control center or doctor/physician if you feel unwell. If skin irritation occurs or eye irritation persists, get medical advice/attention.

REF 317111
REF 317112

CE
0086



Rx ONLY



Inter-Med, Inc.
2200 South St.
Racine, WI 53404, USA
Tel: +1-262-636-9755

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

317111-I-V-08-ENG (4)



EDGEENDO®
Twice as Good. Half the Cost.

1-855-985-3636

EDGEENDO.COM