

# ENDOMAX® BRITE

## Endodontic Dental Handpiece

### INSTRUCTIONS

 **Read Instructions Before Use**

#### INSTALLATION


##### Installing the contra angle mini head:

1. Push the contra angle mini head securely into the Endomax unit.
2. To remove the contra angle mini head, pull gently until it is detached from the handpiece.



##### Installing the LED light module:

1. Insert the LED light module into the connector below the handpiece.
2. Attach the LED light module to the contra angle mini head.

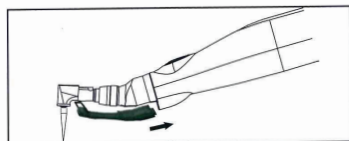


##### Changing the root canal file (latch type):

1. Open the latch by pushing the latch sideways.
2. Insert the root canal instrument into the head of the contra angle and turn until the instrument engages. Verify the file is all the way in.
3. Close the latch in a locked position. Verify the latch is secured with the file in place.

##### Please Note:

During use the endodontic file temperature may reach 43°C, under 40°C ambient temperature.

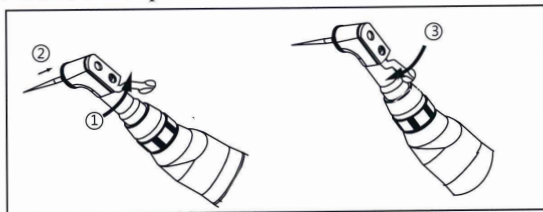

## WARRANTY

### DENTAMERICA®

warrants this unit against defects in material or workmanship for a period of one year from the date of purchase.

This warranty does not apply to damage arising improper operation of the unit.

### DENTAMERICA®

shall repair or replace the defective unit upon return of the unit together with evidence of purchase date to its authorized dealer.

ISO 13485 Certified Manufacture



Date of manufacture



INSYZOR  
76 Windsor Drive Yate Bristol BS37-5DY, England UK



Consult instructions for use



DHEF, INC  
15F, No.106, Hsin Tai Wu Rd., Sec.1,  
Hsi Chih Dist., New Taipei City,  
Taiwan R.O.C.



Refer to instructions



Type BF Applied Part

B1788RT-01  
Rev. 0 6-27-2017

## DENTAMERICA®

18688 E. San Jose Ave.  
City of Industry, CA 91748 U.S.A.  
[WWW.DENTAMERICA.COM](http://WWW.DENTAMERICA.COM)



**EDGEENDO®**  
Twice as Good. Half the Cost.

1-855-985-3636

[EDGEENDO.COM](http://EDGEENDO.COM)



**EDGEENDO®**

# ENDOMAX® BRITE


## Endodontic Dental Handpiece

### INSTRUCTIONS



Read Instructions Before Use

#### CHANGING THE BATTERY

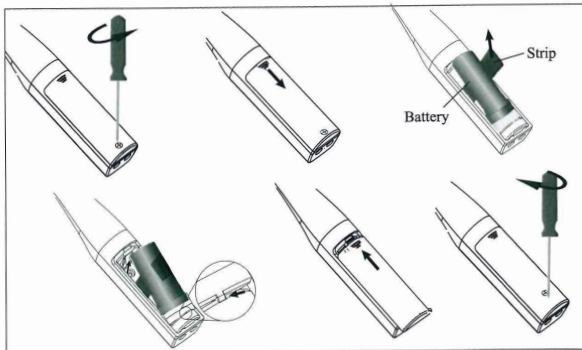

Use a screwdriver to remove the screw on the battery panel. Slide the battery panel downward to remove it. Pull on the strip to help remove the old battery. Insert the new battery and replace the battery panel by sliding it upward. Refasten the screw.

## WARRANTY

### DENTAMERICA®

warrants this unit against defects in material or workmanship for a period of one year from the date of purchase.

This warranty does not apply to damage arising improper operation of the unit.

### DENTAMERICA®

shall repair or replace the defective unit upon return of the unit together with evidence of purchase date to its authorized dealer.

ISO 13485 Certificated Manufacture



Date of manufacture



INSYZOR  
76 Windsor Drive Yate Bristol BS37-5DY, England UK



Consult instructions for use



DHEF, INC  
15F, No.106, Hsin Tai Wu Rd., Sec.1,  
Hsi Chih Dist., New Taipei City,  
Taiwan R.O.C.



Refer to instructions



Type BF Applied Part

B17BBRT-01  
Rev. 01 03/2017

## DENTAMERICA®

18688 E. San Jose Ave.  
City of Industry, CA 91748 U.S.A.  
[WWW.DENTAMERICA.COM](http://WWW.DENTAMERICA.COM)



**EDGEENDO®**  
Twice as Good. Half the Cost.

1-855-985-3636

[EDGEENDO.COM](http://EDGEENDO.COM)