



Gutta percha points are a root canal obturation material. It is used to fill the cleaned, shaped and irrigated root canal space.

- Time saving with easy filling
- For faster and easier root canal obturation
- Variable tapers & sizes
- Excellent radiopacity
- Long shelf life
- Suitable for vertical and lateral condensation

INDICATION AND PURPOSE

1. Gutta percha point is an integral material in promoting periapical healing and preventing disease progression.
2. Endodontic therapy system using the gutta percha point obturated impermeable to liquid and bacteria

INSTRUCTIONS FOR USE

1. Prepare root canal by file.
2. After clean and dry the canal with EDTA solution, apply the sealer into the root canal wall by paper points.
3. Select master gutta percha point matched in prepared root canal.
4. Irrigate a gutta percha point with 5% NaOCl disinfecting solution.
5. Insert master point in to the root canal to working length.
6. Insert accessory points into the root by finger spreader.
7. Confirm a filling by x-ray.

CAUTIONS

1. Do not use this product for any patients with a history of hypersensitivity reactions to it, such as a rash, dermatitis, etc.
2. Some patients have an allergic reaction to Trans 1,4 Polyisoprene materials and/or even experience hypersensitivity. Do not use this product for them.
3. This product is intended only for use for the tissue inside the mouth. Be careful to prevent it from coming into contact with eyes. If material has come into contact with eyes or skin, wash immediately with plenty of water and then consult a doctor.
4. Non-specialists are prohibited from using this product. Do not use this product for any purposes other than its intended purposes.
5. Should be careful to avoid overfilling or dead space.
6. Be sure to take measures for the protection of the respiratory tract before.
7. Always close the slide lid after use.


STORAGE

Store at room temperature of 1°C~30°C(33°F~86°F)

SHELF LIFE

4 years from manufacture date

Symbols

-  Manufacturer
-  Date of manufacture
-  Batch Code
-  Use By
-  Caution
-  Do not re-use
-  Keep away from sunlight
-  Temperature limitation

