INSTRUCTIONS FOR USE

Easier Filing with Lubricating Action

Applications

- Root canal preparation with files and reamers
- Presence of calcification in the canal
- Too narrow canals in posterior teeth
- Smear layer removal

Mode of action

- Removal of inorganic substances by chelation (EDTA - Ca)
- Easier filling due its lubricating action
- Produces the cleanest canal wall after smear layer removal.

Components

EdgeGel 7.0g (5.4ml) x 2
 Disposable tips
 Net weight : 14g

Directions for use

1. Deposit a small amount of EdgeGel in the canal orifice.
2. Carefully pump into the canal with a fine file or reamer.
3. Reapply EdgeGel when necessary or until cleaning and shaping is complete.

How to use Rotary Ring

If in contact with air, content could be hardened, so it is necessary to cover a cap on the needle after use.

Putting a syringe into a rotary ring is more convenient to handle & use the rotary ring as a wrench to tighten or loosen a tip.

Cautions

- Use EdgeGel with discretion to prevent perforation.
- Avoid extrusion of the solution into the apex.
- Keep the syringe capped all the time.

Storage

Store at temperatures 1°C - 30°C (33°F - 86°F)

Shelf life

2 years from the manufactured date.

Symbols

-  Manufacturer
-  Date of manufacture
-  Batch Code
-  Use By
-  Consult instructions for use
-  Keep away from sunlight
-  Temperature limitation
- Rx Only** Caution : Federal law restricts this device to sale by or on the order of a dentist

Distributed by

EDGEENDO[®]

5600 Wyoming Blvd NE Suite 100
 Albuquerque, NM 87109
 Phone: 855.985.3696
 www.edgeendo.com

 Manufacturer **METABIOMED CO.LTD.**
 270, Osongsaengmyeong 1-ro, Osong-eup,
 Heungdeok-gu, Cheongju-si,
 Chungcheongbuk-do, Korea

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