EDGE UTOPIA™

BIOCERAMIC SEALER 2G

ENGLISH

DEFINITION

Injectable endodontic bioceramic sealer ready for use.

INDICATIONS

- Sealing the root canal of permanent teeth;
- Internal reabsorption treatment;

COMPOSITION

Calcium Silicates, calcium aluminate, calcium oxide, zirconium oxide, iron oxide, silicon dioxide and dispersing agent.

TECHNICAL INFORMATION

- Biocompatible product ready for use;
- Setting time: 120-240 minutes (2-4 hours) after insertion into the canal, according to ISO 6876;

Ⓢ Important: Dry with paper cones, but without excessively drying the canal; the moisture from the dentin tubules will initiate the material's setting reaction;

- High alkalinity - pH: 12.5;
- High radiopacity: 7 mm of aluminum scale, according to the ISO 6876 standard;
- No shrink at setting time;
- It can be used, in addition to conventional techniques, single-cone.

USE TECHNIQUE

Use of Applicator Tips

- We recommend using a disposable plastic cover on the syringe to avoid cross-contamination;
- Confirm the material exits from the syringe before direct application to the channel;
- Position the tip and syringe assembly properly into the canal and depress the plunger lightly to prevent excessive output of the product;
- Before closing the syringe, retract the plunger to stop the outflow;
- Clean any excess material on the tip of the syringe and close with appropriate pressure after use to prevent contact with moisture, which causes the product to dry;
- Clean and disinfect the syringe in subsequent uses to avoid cross-contamination; in case of contamination with saliva or blood, discard it.

A. SEALING THE ROOT CANAL OF PERMANENT TEETH;

Traditional technique - Lateral compression

- Anesthetize, install absolute isolation and then perform the biomechanical preparation of the canal;
- Remove the sealing tip and attach the applicator tip on the syringe;
- Dry the canal only with absorbent paper cones without causing excessive dryness;
- Apply EdgeUtopia BioCeramic Sealer in the apical third of the canal directly with the syringe;
- Insert the gutta percha cone in the canal according to the last file used covered with EdgeUtopia BioCeramic Sealer;
- X-ray for checking the correct filling of the canal;
- Cut the cone at the desired height with heated instruments followed by vertical compression;
- Remove, with water, excessive material from the canal walls, perform crown sealing and restoration.

Important:

- EdgeUtopia BioCeramic Sealer does not suffer setting contraction;
- EdgeUtopia BioCeramic Sealer can be used, in addition to conventional techniques, single-cone;
- To use the capsule presentation, place it on the gun, depress the plunger to dispense the material on a glass plate or dispense it directly into the canal, performing one of the techniques mentioned above;
- Use conventional gutta removal techniques.

Single-cone technique

- Anesthetize, install absolute isolation and then perform the biomechanical preparation of the canal;
- Remove the sealing tip and attach the applicator tip on the syringe;

- Dry the canal only with absorbent paper cones without causing excessive dryness;
- Apply EdgeUtopia BioCeramic Sealer in the apical third of the canal directly with the syringe;
- Insert the accessory gutta percha cone;
- X-ray for checking the correct filling of the canal;
- Cut the cones at the desired height with heated instruments followed by vertical compression;
- Remove excessive material from the canal walls with suitable irrigating solutions for this purpose, perform crown sealing and restoration.

B. INTERNAL REABSORPTION TREATMENT

- Anesthetize and install absolute isolation;
- Remove granulation tissue from reabsorption area with sharp curettes;
- Neutralize the area with calcium hydroxide paste;
- Remove the calcium hydroxide in the next session;
- Dry the canal only with absorbent paper cones without causing excessive dryness;
- Insert EdgeUtopia BioCeramic Sealer with the applicator tip across the canal, prioritizing the reabsorption area;
- Insert the accessory gutta percha cones according to the lateral compression technique;
- Cut the cones at the upper part of the reabsorption with heated instruments. Perform vertical compression for better disposal of the sealer in the reabsorption area;
- X-ray for checking the correct filling of the reabsorption and canal area;
- Fill the remaining of the canal with gutta percha;
- Perform crown sealing with glass ionomer or other material of your preference and restore the tooth;
- X-ray and follow up for at least two years

Filling removal

- Use conventional gutta removal techniques.

WARNINGS

- Use absolute isolation in all indications;
- Do not use EdgeUtopia BioCeramic Sealer to fill the canal without the gutta percha cones;
- The product is sensitive to moisture, so properly close the packaging with proper pressure to prevent dryness;
- Never store it in a refrigerator;
- Dry the canal with paper cones, but without excessively drying the canal; the moisture from the dentin tubules will initiate the material's setting reaction.
- Use the product only after controlling the infection, because at this stage the material's setting reaction does not occur due to the acidic pH in the area;
- Avoid causing excessive overflow of the product to the periapical region;
- Do not reuse the applicator tips;
- Clean and sterilize the syringe in subsequent uses to avoid cross-contamination.

PRECAUTIONS

- Wear safety goggles, mask and gloves when handling the product. In case of contact with eyes or skin, wash with water;
- Do not use the product after its expiration date;
- Do not use the product if its packaging is damaged.
- Do not use it in patients that report sensitivity to the components of the formula.
- Product not tested in pregnant and lactating women;
- Keep the product away from children.

STORAGE

- Keep the product in a dry and ventilated place between 15 and 30°C and with relative humidity below 60%.
- Do not store it in a refrigerator.

* Product used exclusively by the dental surgeon

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