


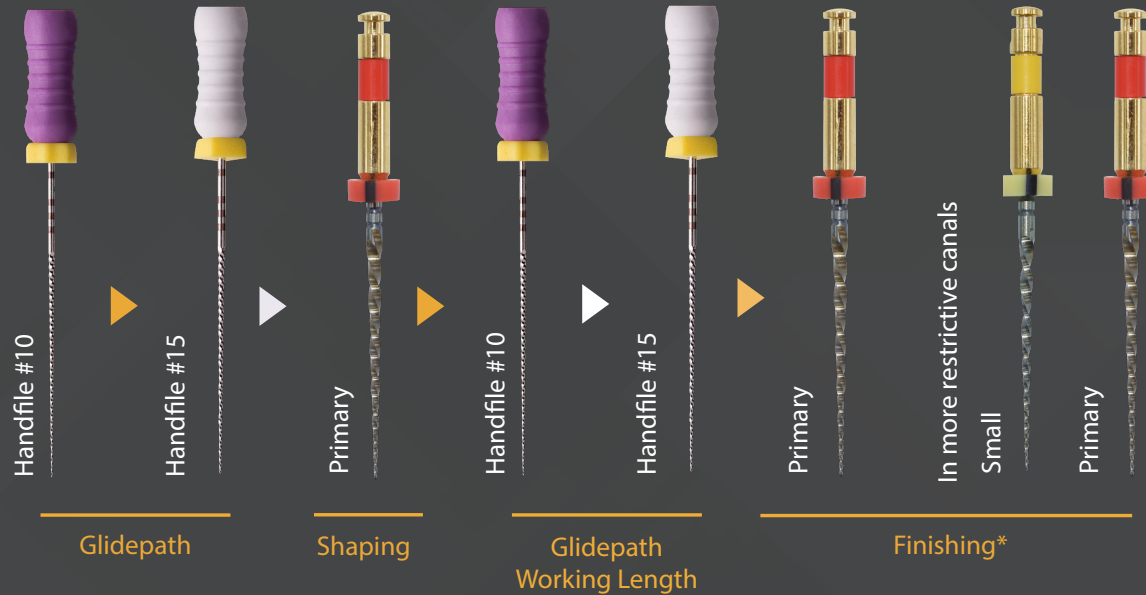

*If the Primary file (#25 Red) reaches working length with minimal resistance or if clinician desires a larger apical shape additional instruments are available: Medium (#35 Green) and Large file (#45 White).

Motor settings:

- The EdgeOne Fire files shape with a reverse reciprocating motion.
- Edge HP Motor setting instruction: Press power button, press and hold P button, press + or – button to find the desired preset setting (ex. EdgeOne Fire B).
- Reciprocation options:
Edge One Fire A (max safe)= F:60/R:150
Edge One Fire B (max efficiency)=F:70/R:180
Edge One Fire C = intelligent reciprocation (CCW-ATR)
- If using an alternative motor the following setting may be used: 350 rpm (reciprocating), Torque 4.0-5.2 Ncm

Protocol for use:

Refer to the written IFU for step by step details of the illustration.

Scan to access:



EDGEENDO®



Fill the canal using a gutta percha cone that matches the size of the last file used. Use a bioceramic sealer to ensure proper sealing of the canal.