


# EDGEONE BLAZE UTOPIA™

Alternative to WaveOne® Gold

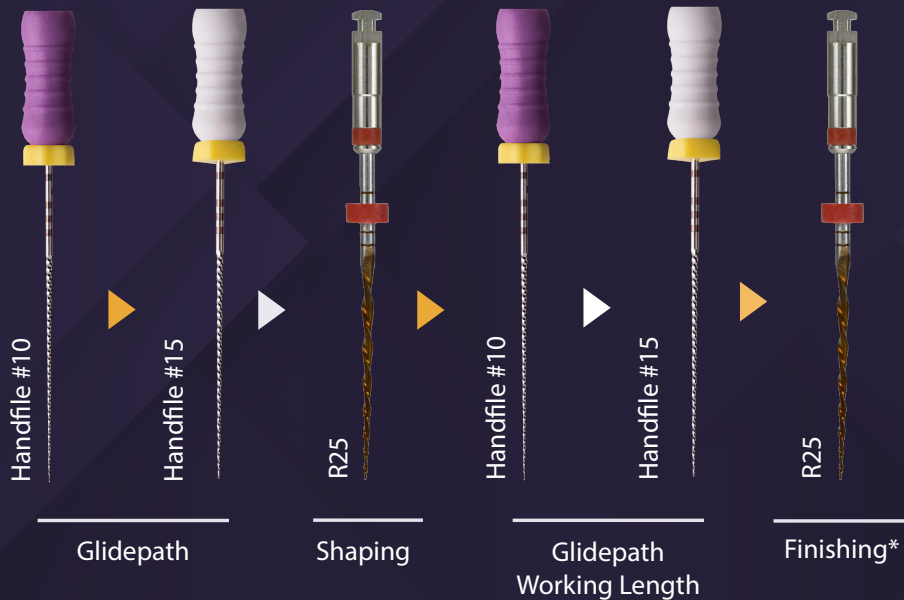

\*If the R25 file does not reach working length, utilize the R20 file in one or more passes to reach working length.

If either the R20 or R25 file is loose at working length without dentinal debris in the apical flutes, continue shaping with the next larger file in the sequence R35 or R45—until the apical flutes are adequately loaded with clean debris.


\*WaveOne® Gold is a registered trademark of Dentsply Sirona.

## Motor settings:

- The EdgeOne Blaze Utopia files shape with a counterclockwise reciprocating motion.
- Edge HP Motor setting instruction: Press power button, press and hold P button, press + or – button to find the desired preset setting (ex. EdgeOne Fire B).
- Reciprocation options:  
Edge One Fire A (max safe)= F:60/R:150  
Edge One Fire B (max efficiency)=F:70/R:180  
Edge One Fire C = intelligent reciprocation (CCW-ATR)
- If using an alternative motor the following setting may be used: 170°CCW/60°CW

## Protocol for use:

Refer to the written IFU for step by step details of the illustration.  
Scan to access:

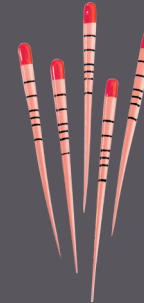

Fill the canal using a gutta percha cone that matches the size of the last file used. Use a bioceramic sealer to ensure proper sealing of the canal.