


EDGETAPER BLAZE UTOPIA™

Alternative to ProTaper Gold®

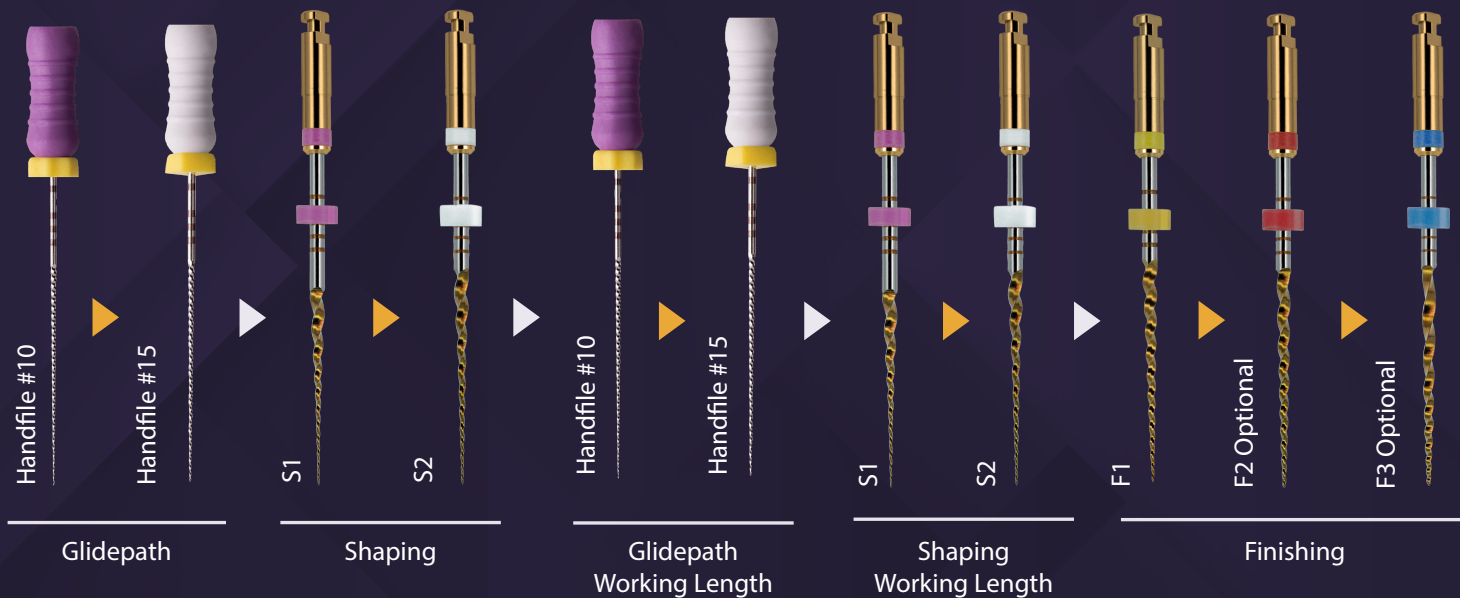


*ProTaper® Gold is a registered trademark of Dentsply Sirona.

Motor settings:

- Use files in canals with a confirmed, reproducible glide path.
- Use in a deliberate brushing motion on the out-stroke.
- Irrigate, recapitulate with a #10 file and re-irrigate after removing each rotary file.
- Frequently clean and inspect the flutes of the rotary file.
- Use SX to eliminate triangles of dentin when present.
- Use at 300-450 RPM with a torque of 5.2 Ncm.

Protocol for use:

Refer to the written IFU for step by step details of the illustration.
Scan to access:

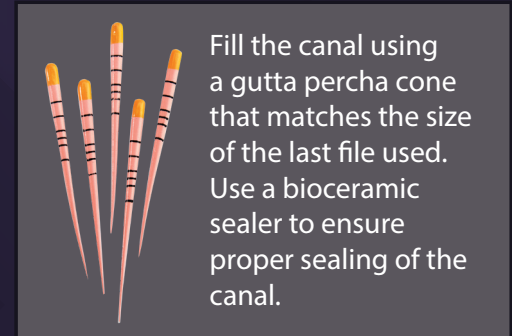

Fill the canal using a gutta percha cone that matches the size of the last file used. Use a bioceramic sealer to ensure proper sealing of the canal.