



EDGEENDO®

Sodium Hypochlorite

The container may be stored at room temperature, but it is recommended to store at 2°-8°C (36°-46°F) to increase stability. Shelf life is 30 months from date of manufacture, if stored properly. Storage at room temperature may reduce shelf life by up to 6 months. Refrigeration after opening is recommended.

Description

EdgeEndo® Sodium Hypochlorite is a Sodium Hypochlorite solution in water with wetting agents to lower surface tension. The solution is a root canal cleanser for use in endodontic procedures.

Instructions for use

Important: Set rubber stops on irrigation needle tips a minimum 2mm short of working length. Avoid wedging irrigation needle tip into root canal and excessive pressure during irrigation.


Syringes

1. Remove cap from Red syringe.
2. Attach irrigation needle tip into the syringe.
3. Irrigate canal.
4. Discard irrigation needle tip & syringe after use.

Bottle

1. Select desired size syringe and irrigation tip.
2. Fill syringe with EdgeEndo® Sodium Hypochlorite.
3. Attach irrigation needle tip onto the syringe.
4. Irrigate canal.
5. Discard irrigation needle tip & syringe after use.

Distributed By:
EdgeEndo®
Albuquerque, NM 87109
1-855-985-3636
EdgeEndo.com
Made in the USA

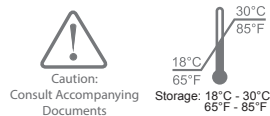

Rx only
For Dental Use Only