EDGE  **UTOPIA RRM™**

Dispensing Gun

CHOOSE YOUR LANGUAGE

ENGLISH	3	ITALIANO	21
FRANÇAIS	4	LIETUVIŠKAI	22
ESPAÑOL	5	LATVIEŠU	24
PORTUGUÊS	7	POLSKI	25
HRVATSKI	8	ROMÂNĂ	27
ČESKY	10	SLOVENČINA	28
DANSK	11	SLOVENSKY	29
NEDERLANDS	12	SVENSKA	31
EESTI KEEL	14	NORSK	32
SUOMI	15	БЪЛГАРСКИ	33
DEUTSCH	17	СРПСКИ	35
TÜRKÇE	18	ΕΛΛΗΝΙΚΑ	36
MAGYAR	20		

ENGLISH

DEFINITION

System for application of dental materials.

INDICATIONS

Suitable for application of dental materials such as cements, resins, pastes, gels and single-dose devices when supplied in EDGEUTOPIA RRM™ DISPENSING GUN compatible capsule form for dental procedures.

COMPOSITION

Body: Polycarbonate

Spring: Stainless steel

WARNINGS

Note: Any serious incident related to the device must be reported to the manufacturer and the competent health authority in the country where the user and/or patient are located

PRECAUTIONS

Use only compatible capsules.

Capsules must not be reused. Discard after use.

TECHNIQUE FOR USE

CAPSULE

- Disinfect the capsule and the EDGEUTOPIA RRM™ DISPENSING GUN with Alcohol 70%.
- Before applying the product, verify the capsule containing the product is not damaged or defective.
- Insert the capsule into the EDGEUTOPIA RRM™ DISPENSING GUN and remove the cap.
- Press the EDGEUTOPIA RRM™ DISPENSING GUN handle using slow and uniform pressure, to extrude the appropriate amount of material from the capsule.
- Remove the required amount of product and apply with the appropriate instrument to the area to be treated.
- Important: For easily accessible areas, the product can be applied directly from the capsule using the EDGEUTOPIA RRM™ DISPENSING GUN.
- Insert the material into the treated area.

- Finalize the treatment according to the appropriate methods.
- Remove the capsule and discard it after use.
- Perform cleaning and disinfection of the EDGEUTOPIA RRM™ DISPENSING GUN by an appropriate method.
- Ensure that the capsule is at the end of the EDGEUTOPIA RRM™ DISPENSING GUN for easier removal.

STORAGE

Store in a cool dry place.

DISPOSAL METHOD

Dispose of capsules after use, in accordance with national laws and recommendations for proper disposal from local authorities.

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