INSTRUCTIONS FOR USE – EDGESEQUEL SAPPHIRE

EN	INSTRUCTIONS FOR USE	2
BG	ИНСТРУКЦИИ ЗА УПОТРЕБА	6
CS	NÁVOD K POUŽITÍ	10
DA	BRUGSANVISNING	14
DE	GEBRAUCHSANWEISUNG	18
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ.....	22
ES	INSTRUCCIONES DE USO	26
ET	KASUTUSJUHEND.....	30
FI	KÄYTTÖOHJEET	34
FR	NOTICE D’UTILISATION.....	38
HR	UPUTE ZA UPORABU	42
HU	HASZNALATI UTASITAS.....	46
IT	ISTRUZIONI PER L’USO	50
LT	NAUDOJIMO INSTRUKCIJA	54
LV	LIETOŠANAS INSTRUKCIJAS	58
NL	GEBRUIKSAANWIJZING	62
NO	BRUKSANVISNING.....	66
PL	INSTRUKCJA UŻYWANIA.....	69
PT	INSTRUÇÕES DE UTILIZAÇÃO	73
RO	INSTRUCȚIUNI DE UTILIZARE	77
SK	NÁVOD NA POUŽITIE.....	81
SL	NAVODILA ZA UPORABO	85
SV	BRUKSANVISNING.....	89
TR	KULLANIM TALIMATLARI	93

INSTRUCTIONS FOR USE

EN


FOR DENTAL USE ONLY

INSTRUCTIONS FOR USE – EDGESEQUEL SAPPHIRE

1) GENERAL DESCRIPTION

Composition:

The instrument is made of an Annealed Heat Treated (Fire-Wire™) nickel-titanium alloy and an endo stop made of silicone. All files have constant taper and delivered in sterile state (sterilized by irradiation).

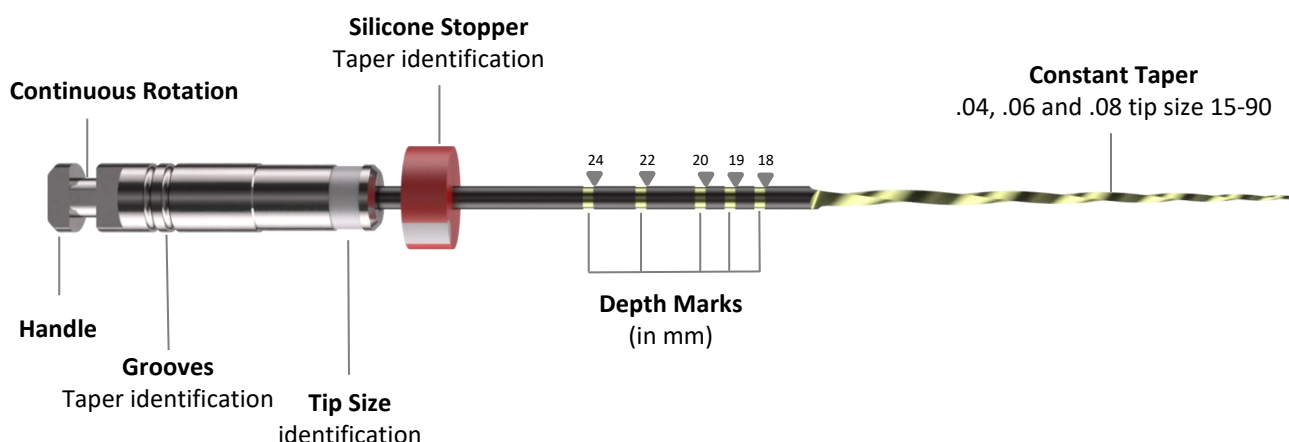Tip size and taper are identified using two color marks: the colored ring on the handle indicates the tip diameter, and the stopper color indicates the taper.

015	020	025	030	035	040
045	050	055	060	070	080
090					

The grooves in the shank type indicates the taper: .04, .06 and .08. The handle length is 11 mm.

2 grooves = .04

3 grooves = .06

4 grooves = .08

Parameters for use:

These instruments are compatible with endodontic motors compliant with ISO 1797.

Recommended speed: 500 rpm continuous rotation; torque limit: 408–530 g·cm (4.0–5.2 N·cm).

2) INTENDED USE

The EdgeSequel Sapphire™ endodontic instruments are intended for root canal treatment.

3) INDICATIONS FOR USE

EdgeSequel Sapphire™ are indicated in case of pulp or periapical diseases that require root canal treatment. All instruments are designed for preparation and shaping of root canals.

4) CONTRAINDICATIONS AND ADVERSE REACTIONS

None known.

5) WARNINGS AND PRECAUTIONS

- These instruments have not been tested on children, pregnant nor breastfeeding women.
- These endodontic instruments must not be used on individuals with a known allergic sensitivity to nickel-titanium and silicone.
- Single use instruments must not be retreated for subsequent use. This could compromise their integrity.
- Each instrument may be used to treat up to four (4) canals during a single procedure. Discard after use.
- For your own safety, wear personal protective equipment (masks, gloves, and goggles) during the treatment and reprocessing of the instrument.
- Do not use if sterile packaging seal is broken, damaged or wet.
- Do not use the instruments after expiration date.
- Reprocess the instrument appropriately before disposal.
- Carefully read the label, marking on the packaging and this instruction for use to ensure the correct identification and use of device.
- Use a dental dam when using endodontic instruments to avoid inhalation or ingestion by the patient.
- Take several radiographs from different angles to determine the anatomy of the root canals (length, width and curvature).
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- Cracks, deformations, signs of corrosion, loss in color or marking are signs that the device is no longer able to achieve the required performance level and should be discarded.
- Regularly clean flutes and inspect instrument during use, and discard if it shows any signs of distortion or wear.
- The packaging of the instruments should be carefully examined before opening (packaging integrity, no humidity, no visual degradation of instruments) to ensure that the packaging's integrity has not been compromised during transport and storage.
- If the packaging has been opened, damaged, become wet or the expiry date has passed, the sterile state of the instruments inside the packaging is not guaranteed. Discard the device(s) that are single use and consult the reprocessing instructions available on EdgeEndo's website for multiple use device(s).
- All serious events occurring in relation to the product must be reported to the manufacturer and the competent authority according to local regulations.

6) PERFORMANCE CHARACTERISTICS

EdgeSequel Sapphire™ rotary files are compliant with all applicable testing requirements of the ISO 3630 series of standards.

7) PROTOCOL FOR USE

1) Access and Canal location:

Create straight-line access to canal orifices. Locate canals and explore using stainless steel hand instruments.

2) Irrigation:

Irrigate the canal with an appropriate irrigating solution (e.g. sodium hypochlorite or EDTA) before each hand or rotary instrument.

3) Working Length and Glidepath:

- Determine working length using an apex locator and radiographs.
- Establish a glidepath to working length with a stainless steel ISO #10 file or larger.
- A mechanical glidepath instrument may be used according to the manufacturer's instructions.

4) Canal Shaping Using EdgeSequel Sapphire™:

Shape the canal using EdgeSequel Sapphire™ according to the anatomy of the canal:

- EdgeSequel Sapphire .04 taper: Suitable for shaping all canals.
- EdgeSequel Sapphire .06 taper: Recommended for straight or larger canals.

Crown-Down technique (EdgeSequel Sapphire .04 or EdgeSequel Sapphire .06):

- Start with 25/04 or 25/06 and advance to resistance or working length (whichever occurs first).
- If resistance is encountered before reaching working length, switch to 20/04 or 20/06 and advance to working length.
- Alternate between sizes (20 ↔ 25 of same taper) until one instrument reaches working length.
- If a larger apical preparation is desired, continue with the next larger size (30, 35, 40, etc.) of the same taper to working length.

Between rotary instruments, recapitulate with a small hand file (ISO #10 or #15) to maintain patency and facilitate irrigant penetration.

5) Final Irrigation and Drying:

Perform a final irrigation protocol according to clinical preference. Dry the canal with paper points that correspond to the final file size taken to working length.

6) Obturation:

Obturate the canal system using a matching gutta-percha cone or thermal carrier system.

8) CONDITIONS AND TERM OF STORAGE, SHELF-LIFE

The shelf-life is 5 years for sterile instruments. The use-by date is indicated on the packaging. Sterility is preserved and guaranteed for the whole shelf life unless the packaging is broken, or the storage conditions are not respected.

9) DISPOSAL

When a device reaches the end of its life, make sure that it is discarded in accordance with the applicable laws and regulations.

10) SYMBOLS

For explanation of symbols for IFUs and labels, please consult the document “Glossary of the symbols used by EdgeEndo” available on EdgeEndo’s website.

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