INSTRUCTIONS FOR USE – EdgeFile® X3

EN	INSTRUCTIONS FOR USE	2
BG	ИНСТРУКЦИИ ЗА УПОТРЕБА	5
CS	NÁVOD K POUŽITÍ	9
DA	BRUGSANVISNING	12
DE	GEBRAUCHSANWEISUNG	15
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ.....	18
ES	INSTRUCCIONES DE USO	22
ET	KASUTUSJUHEND.....	26
FI	KÄYTTÖOHJEET	29
FR	NOTICE D’UTILISATION.....	32
HR	UPUTE ZA UPORABU	35
HU	HASZNALATI UTASITAS.....	38
IT	ISTRUZIONI PER L’USO	41
LT	NAUDOJIMO INSTRUKCIJA	44
LV	LIETOŠANAS INSTRUKCIJAS	47
NL	GEBRUIKSAANWIJZING	50
NO	BRUKSANVISNING.....	53
PL	INSTRUKCJA UŻYWANIA.....	56
PT	INSTRUÇÕES DE UTILIZAÇÃO	60
RO	INSTRUCȚIUNI DE UTILIZARE	63
SK	NÁVOD NA POUŽITIE.....	66
SL	NAVODILA ZA UPORABO	69
SV	BRUKSANVISNING.....	72
TR	KULLANIM TALIMATLARI	75

INSTRUCTIONS FOR USE

EN


FOR DENTAL USE ONLY

INSTRUCTIONS FOR USE – EDGEFILE® X3


1) GENERAL DESCRIPTION

Composition:

The instrument is made of an Annealed Heat Treated (Fire-Wire™) nickel-titanium alloy and an endo stop made of silicone. All files have constant taper and delivered in sterile state (sterilized by irradiation).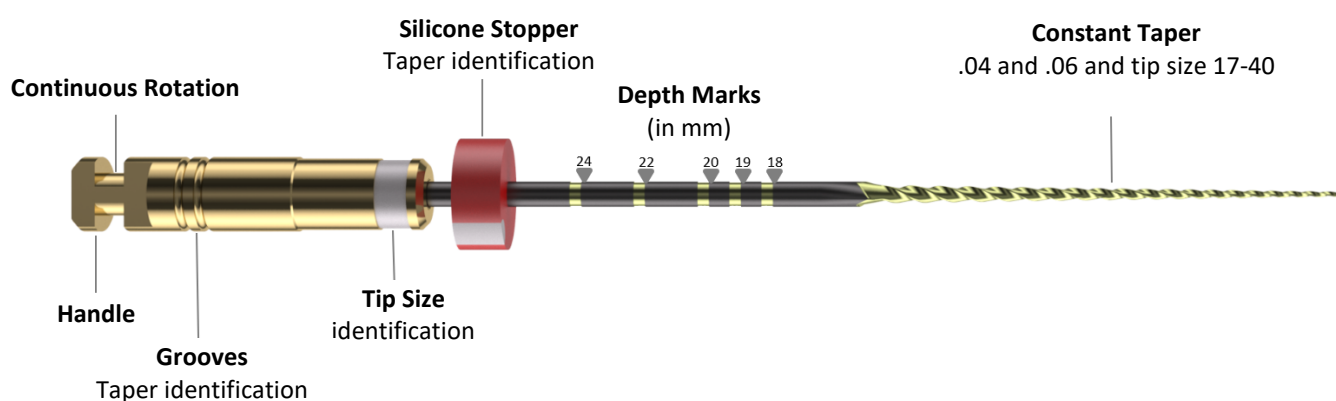

Tip size and taper are identified using two color marks: the colored ring on the handle indicates the tip diameter, and the stopper color indicates the taper.

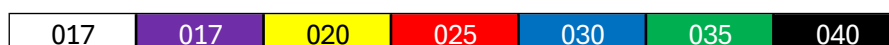The grooves in the shank type indicates the taper: .04 or .06. The handle length is 11 mm:

2 grooves = .04

3 grooves = .06

Parameters for use:

These instruments are compatible with endodontic motors compliant with ISO 1797.

Recommended speed: 300–500 rpm continuous rotation; torque limit: 300 g·cm (2.94 N·cm).

2) INTENDED USE

The EdgeFile® X3 endodontic instruments are intended for root canal treatment.

3) INDICATIONS FOR USE

EdgeFile® X3 are indicated in case of pulp or periapical diseases that require root canal treatment. All instruments are designed for preparation and shaping of root canals.

4) CONTRAINDICATIONS AND ADVERSE REACTIONS.

None known.

5) WARNINGS AND PRECAUTIONS

- These instruments have not been tested on children, pregnant nor breastfeeding women.
- These endodontic instruments must not be used on individuals with a known allergic sensitivity to nickel-titanium and silicone.
- Single use instruments must not be retreated for subsequent use. This could compromise their integrity.
- Each instrument may be used to treat up to four (4) canals during a single procedure. Discard after use.
- For your own safety, wear personal protective equipment (masks, gloves, and goggles) during the treatment and reprocessing of the instrument.
- Do not use if sterile packaging seal is broken, damaged or wet.
- Do not use the instruments after expiration date.
- Reprocess the instrument appropriately before disposal.
- Carefully read the label, marking on the packaging and this instruction for use to ensure the correct identification and use of device.
- Use a dental dam when using endodontic instruments to avoid inhalation or ingestion by the patient.
- Take several radiographs from different angles to determine the anatomy of the root canals (length, width and curvature).
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- Cracks, deformations, signs of corrosion, loss in color or marking are signs that the device is no longer able to achieve the required performance level and should be discarded.
- Regularly clean flutes and inspect instrument during use, and discard if it shows any signs of distortion or wear.
- The packaging of the instruments should be carefully examined before opening (packaging integrity, no humidity, no visual degradation of instruments) to ensure that the packaging's integrity has not been compromised during transport and storage.
- If the packaging has been opened, damaged, become wet or the expiry date has passed, the sterile state of the instruments inside the packaging is not guaranteed. Discard the device(s) that are single use and consult the reprocessing instructions available on EdgeEndo's website for multiple use device(s).
- All serious events occurring in relation to the product must be reported to the manufacturer and the competent authority according to local regulations.

6) PERFORMANCE CHARACTERISTICS

EdgeFile® X3 rotary files are compliant with all applicable testing requirements of the ISO 3630 series of standards.

7) PROTOCOL FOR USE

Simplified Technique Guide

- 1) Create straight-line access to canal orifices.
- 2) Locate canals and explore using stainless steel hand instruments.
- 3) Irrigate before each hand or rotary file with liquid NaOCl or 17% EDTA.
- 4) Determine working length using a #10 or larger stainless steel file with apex locator and/or radiographs and form a reproducible glide path.
- 5) Begin shaping with the first X3 file recommended for the canal anatomy (example 20/04). Use gentle in-and-out motions with light apical pressure until working length is reached.
- 6) If additional enlargement is desired, continue with the next X3 instrument size (example 25/04, 30/04, 35/04, etc.) until the desired apical size is achieved.
- 7) Between each file, recapitulate with a #10 or #15 hand file to maintain glide path and facilitate irrigation.

Canal Preparation:

- 1) Perform steps 1 through 4 in the Simplified Technique Guide, recapitulate with a #10 or #15 hand file to maintain patency and facilitate irrigation flow.
- 2) Shape and clean the root canal using the sequential technique with the appropriate X3 file sizes based on canal anatomy.

Obturation of Canal Systems:

- Match the master cone to the largest X3 file taken to working length.
- If the corresponding GP cone does not reach working length, select the next smaller cone size.

8) CONDITIONS AND TERM OF STORAGE, SHELF-LIFE

The shelf-life is 5 years for sterile instruments. The use-by date is indicated on the packaging. Sterility is preserved and guaranteed for the whole shelf life unless the packaging is broken, or the storage conditions are not respected.

9) DISPOSAL

When a device reaches the end of its life, make sure that it is discarded in accordance with the applicable laws and regulations.

10) SYMBOLS

For explanation of symbols for IFUs and labels, please consult the document “Glossary of the symbols used by EdgeEndo” available on EdgeEndo’s website.

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